


JOURNEY SERIESTM

ALUMINUM THRESHOLD RAMP INSTRUCTIONS AND WARNINGS

Thank you for purchasing a Journey SeriesTM Threshold Ramp from National Ramp!
Please read the following instructions completely before use.

To keep the ramp stable and prevent it from dislocating, you must secure the threshold ramp to the home.

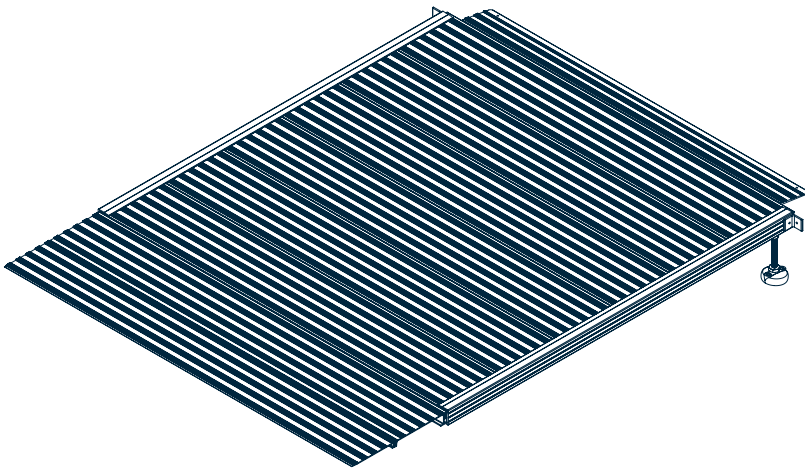

ADJUSTABLE LEG HEIGHT

1. Align the threshold against the door. Check that the TTP properly fits at the door and confirm its placement before snapping into place.




The TTP cannot be repositioned and must be cut to be removed once installed to the threshold ramp.

2. If the adjustable leg needs to be cut to achieve the correct height, place the nut past the cut mark. After cutting, remove nut to rethread bolt.
3. Affix the threshold ramp to the home by screwing down the TTP with low-profile screws appropriate for the material of the home. If the TTP is not being used, a 90° bracket can be used to connect the threshold to the home.

FIXED LEG HEIGHT

1. Affix the threshold ramp to the home by using screws through the side tabs of the fixed leg height threshold.

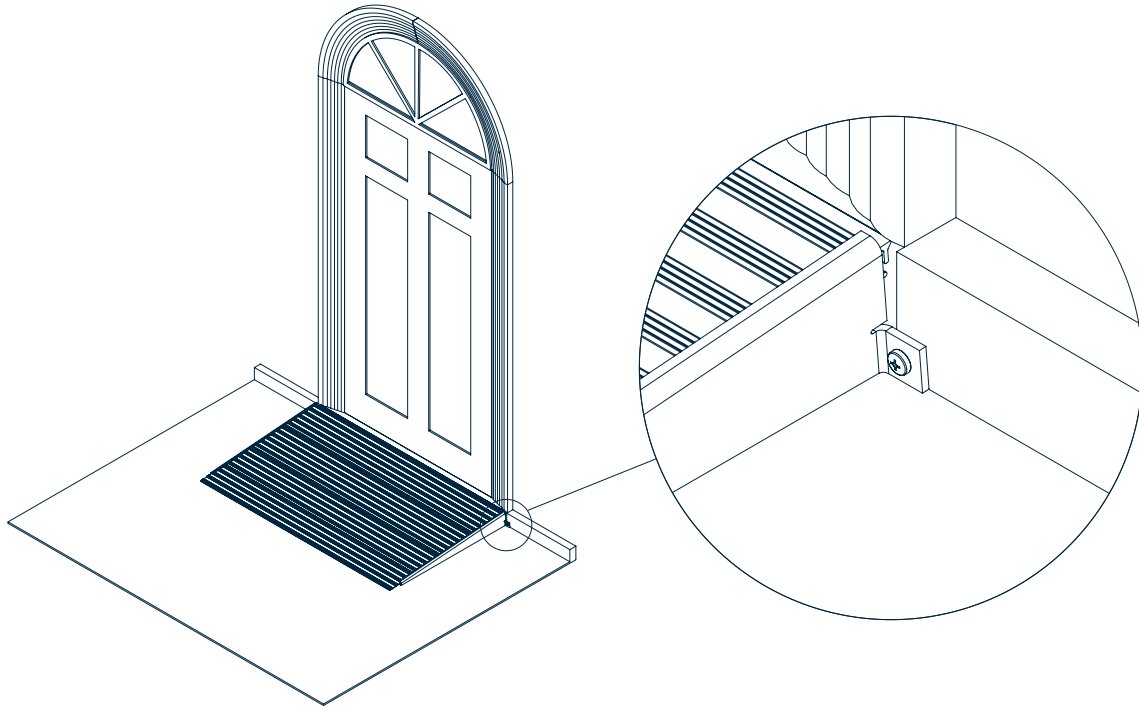

WARNINGS

- Weight capacity of the threshold ramp is 1,000 lbs of evenly distributed weight load.
- Ensure the ramp is fully supported on the ground.
- Only use on secure and stable surfaces.
- While the ramp has a traction surface, use the ramp with caution, especially in slippery conditions.
- Use the ramp with a qualified helper when appropriate.
- Follow the manufacturer's guidelines of your mobility device when using the ramp.
- This ramp has been designed for personal mobility use only.
- This ramp is not a toy. Do not allow children to play on or around the ramp.
- Keep the ramp free from debris.